



How to safely clean your reusable face shield



Perform hand hygiene **BEFORE** and **AFTER** cleaning your reusable face shield



Saturate a clean cloth with a disinfectant solution or wipe
Do not use bleach



Wipe face shield **INSIDE** then **OUTSIDE** with the cloth



Air dry thoroughly in a safe place before storing or using again



Store in a clean zip lock bag or container once dry, when not in use

To improve visibility through your reusable face shield:
use water to remove residue or wipe with an alcohol wipe

Face shields are high-risk surfaces, which can become contaminated with coronavirus (COVID-19).

Clean and disinfect your own reusable face shield before and after use.

Avoid touching the front of your face shield or other eye protection.

To receive this in another format phone 1300 651 160 using the National Relay Service 13 36 77 if required, or email Emergency Management Communications <covid-19@dhhs.vic.gov.au>

Authorised and published by the Victorian Government, 1 Treasury Place, Melbourne.

© State of Victoria, Australia, Department of Health and Human Services. (2001628)_v6_250121

Available at: DHHS.vic – coronavirus (COVID-19) <<https://www.dhhs.vic.gov.au/infection-prevention-control-resources-covid-19>>